



Forever Medical

Portable Dry Biochemistry Analyzer YJ-C2800

Analyzer Feature



45ul Blood



Disposable consumable



≤3 mins testing time



4.3' touch screen



Built-in thermostatic control module



Built-in lithium battery

45ul Blood

Disposable consumable

≤3 mins testing time

4.3' touch screen

Built-in thermostatic control module

Built-in lithium battery



Test Cards



Lipids+Glucose			
Measurement:	TC	2.59~12.93 mmol/L	Sample: Serum, whole blood, plasma
	TG	0.51~7.34 mmol/L	Sample Volume:45μL
	HDL-C	0.39~2.59 mmol/L	Testing time:3 mins
	GLU	2.0~18.0 mmol/L	

Screening for Metabolic diseases			
Measurement:	TC	2.59~12.93 mmol/l	Sample: Serum, whole blood, plasma
	UA	120~1200 mmol/L	Sample Volume: 45μL
	GLU	2~18 mmol/L	Testing time: 3 mins


Analyzer Specification

Analyzer Specifications			
Detection method	Reflection spectrophotometry	Display	4.3' touch screen
Sample type	Factory calibrated needs no further calibration	Data storage	10000 results
Detection time	≤3mins	Operation environment	Temp: 15℃~35℃; Rh: ≤80%
Dimension	25*60*140mm	Storage environment	Temp:-20℃~60℃;Rh: 10%~90%
Weight	220g	Power supply	Bi2Y1A8Y4.50/60



Forever Medical

Easy to use

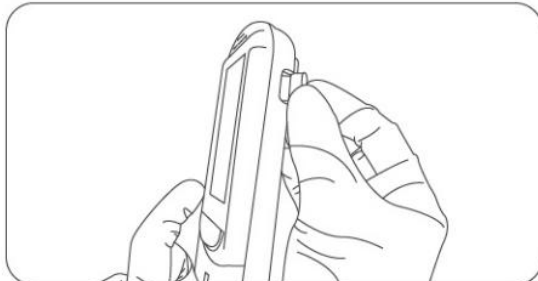


1 Insert the Code Chip with the same bath number into the analyser. Select the sample type to be tested in Setting interface. The default is whole blood.



2 Short press the measurement key and insert the test card.



3 Use a pipette to draw enough blood until the blood reaches the black mark of the pipette.

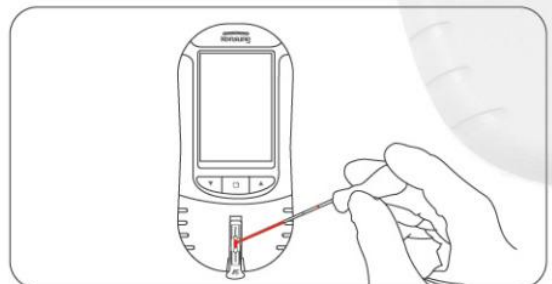

4 Plug the small hole in the red rubber with the index finger, and gently press to drop the blood into the sample area of the test card. After adding the sample, wait for about 3 minutes to display the test result.

HENAN FOREVER MEDICAL CO.,LTD.

Room 806, Floor 8, Wanda Center, Jinshui District, Zhengzhou, Henan Province, China

FAX:86-371-61315853

Email: Julianne@yjcompany.com

Whatsapp/TEL.: 0086-18638789724

Web: www.forermedchina.com www.forevermedical-yj.com